



MODEL: WDTP-10

Instruction Manual



Operator needs to read operation manual and familiar with the operation before using. Retain the manual for future reference.


STRUCTURES

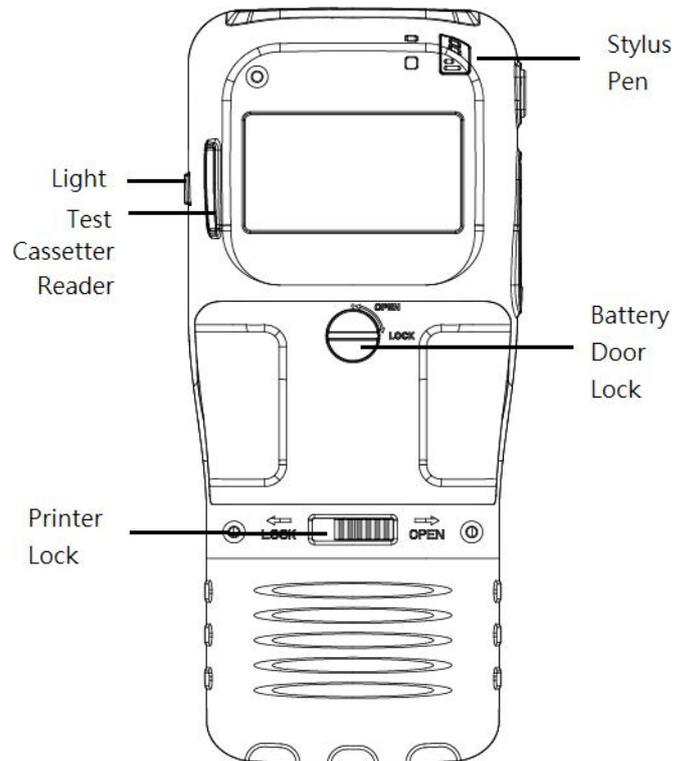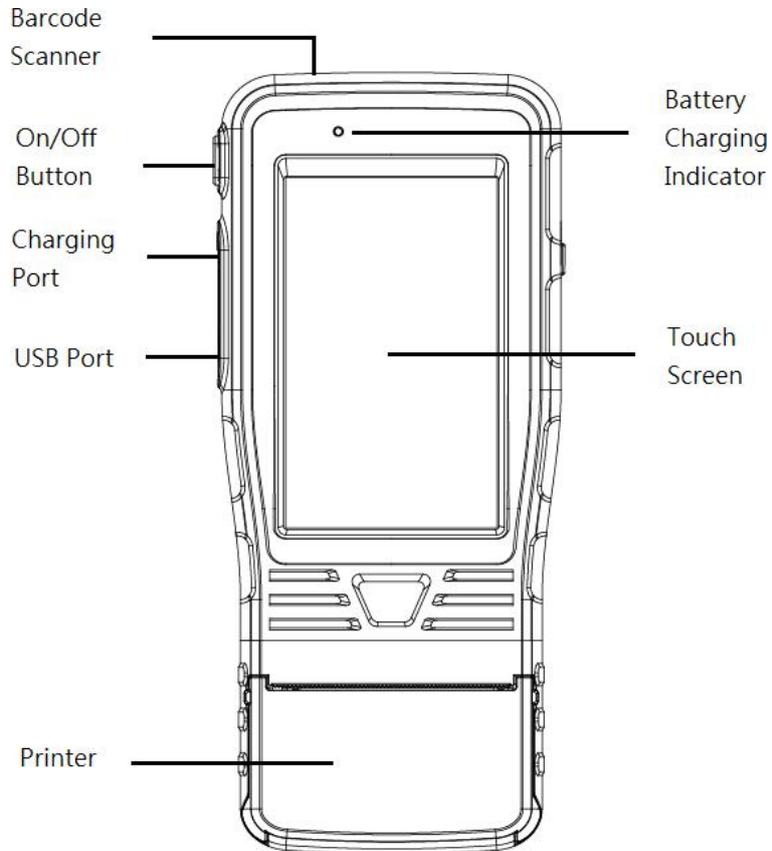

OPERATING INSTRUCTIONS

Test

1. Wear a pair of gloves to do the test.
2. Open a new test pack, remove the test cassette and the sample collector.

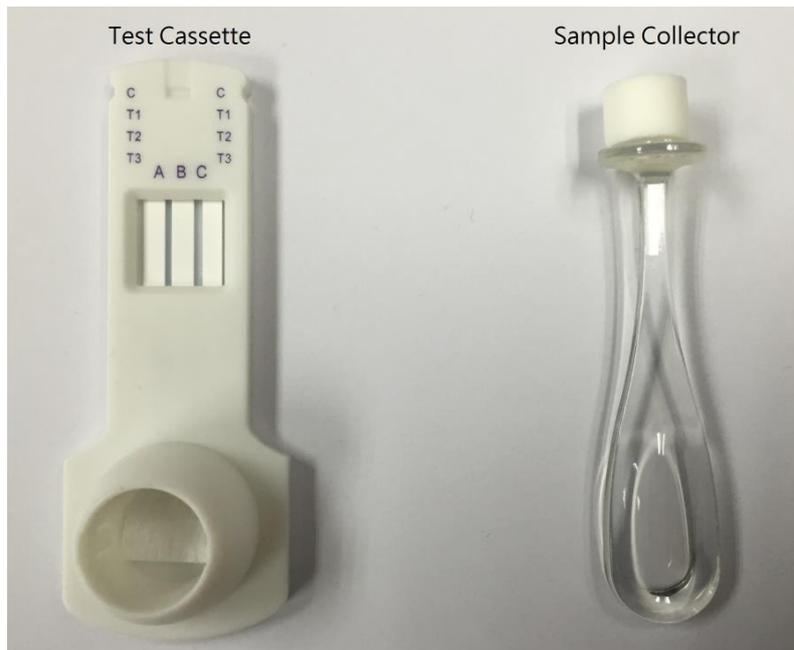


3. Switch On the device.
4. Press “Test” icon to start a new test.





5. Enter the information of the test.
 - ✧ Empty fields will not appear on the printout.

6. Scan the bar code label of the test Cassette.

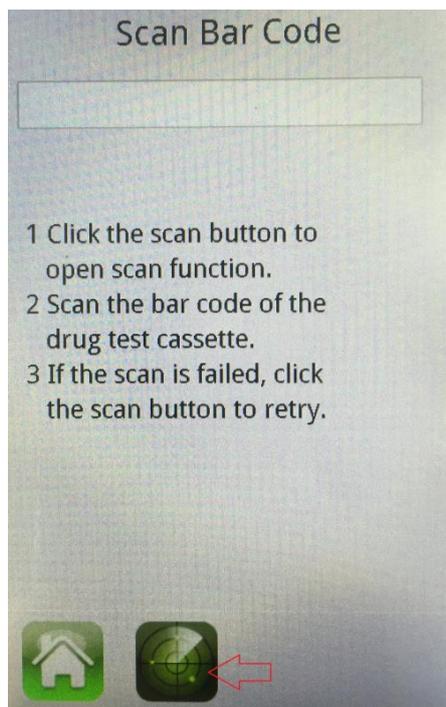

7. Details of the test cassette will be displayed. Press arrow to continue.

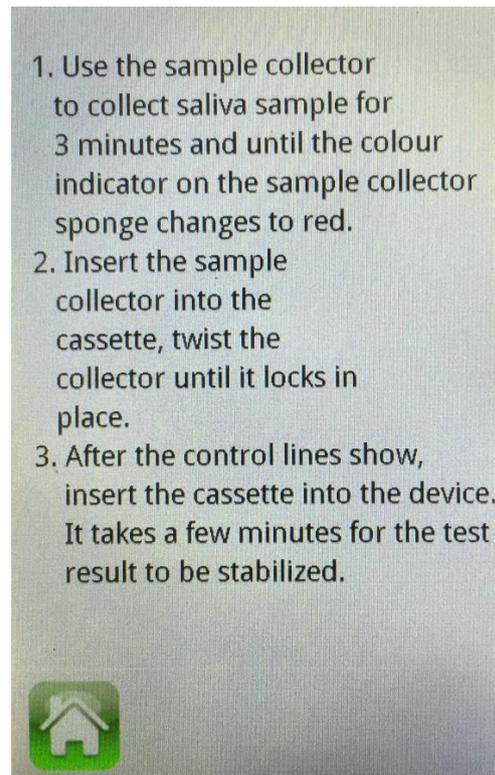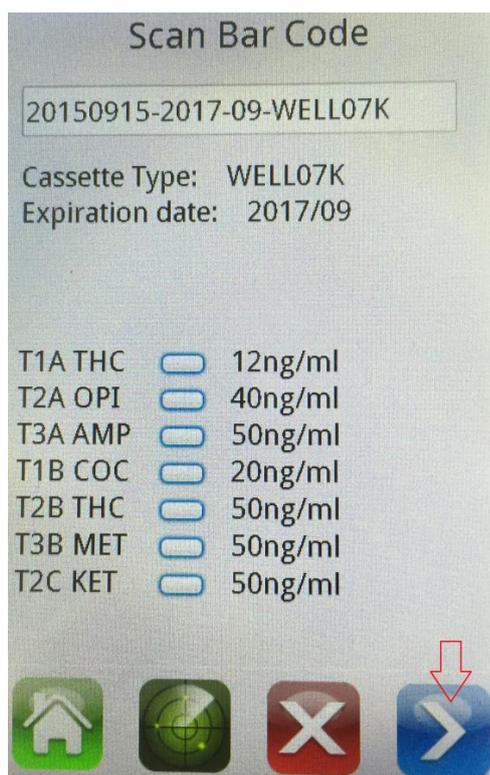

8. Place the sample collector sponge into the testee's mouth, ensure the testee wipes the inside of the mouth, cheeks, cavities and surface of the tongue, then place the sponge under the tongue. Leave it in the mouth for 3 minutes and until the colour indicator on the sample collector sponge changes to red.





9. Insert the sample collector into the test cassette as shown below, twist the sample collector until it locks in place.




10. Wait until the control lines showed, insert the cassette into the device.



11. Once the cassette is inserted, the device will start reading the results. It takes a few minutes for the test result to be stabilized, please wait (press Skip button will display the result immediately, but this is NOT recommended). After the result stabilized, it takes a few seconds to confirm the result.

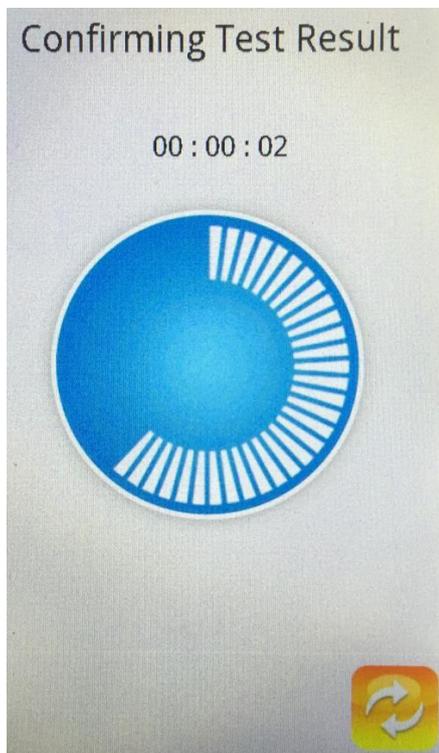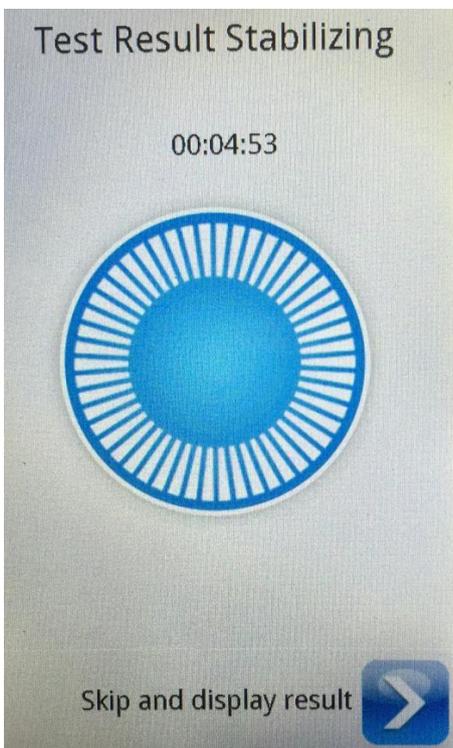


12. The results will be shown on the screen. Press arrow to continue.

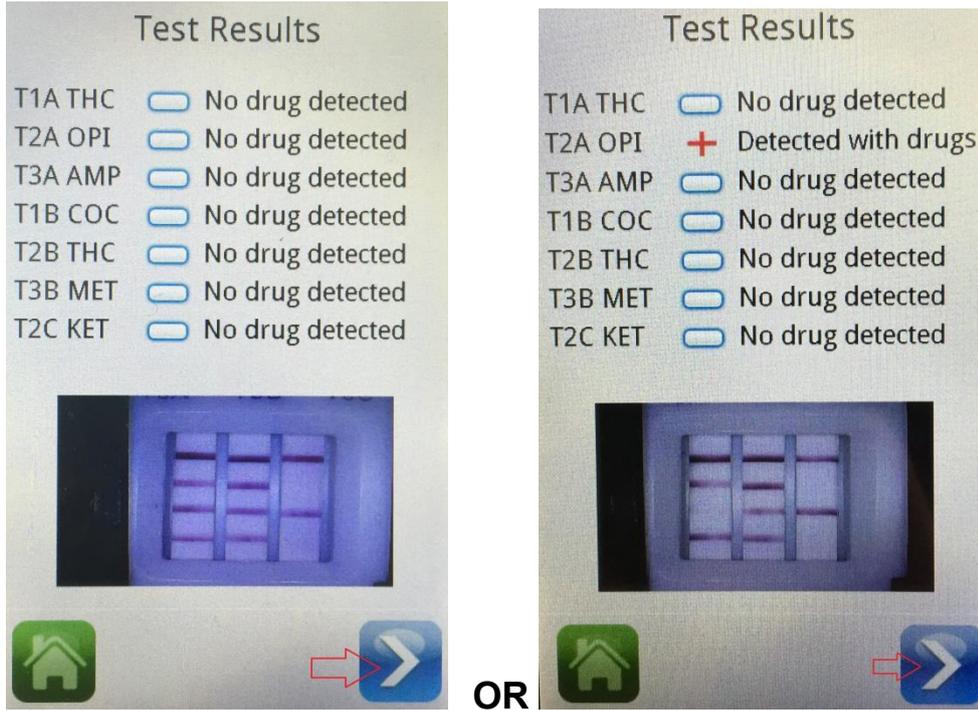


13. If drugs are detected, please arrange for lab test.

14. If the test cassette fails to work or control line does not show, the test is "invalid". Repeat the test with a new test cassette.

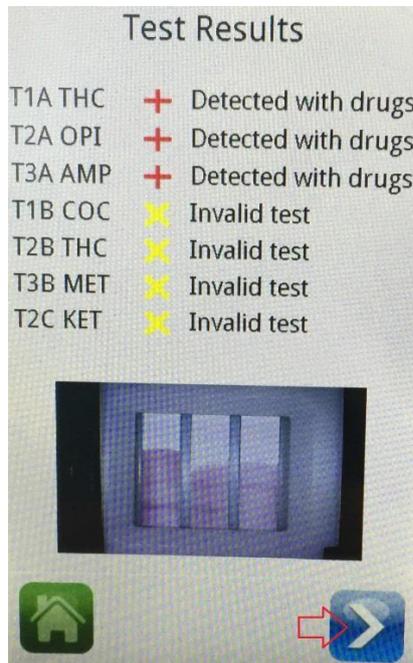


15. If there is an error message “Calibration Error” on the screen, please refer to Function “Verification”.

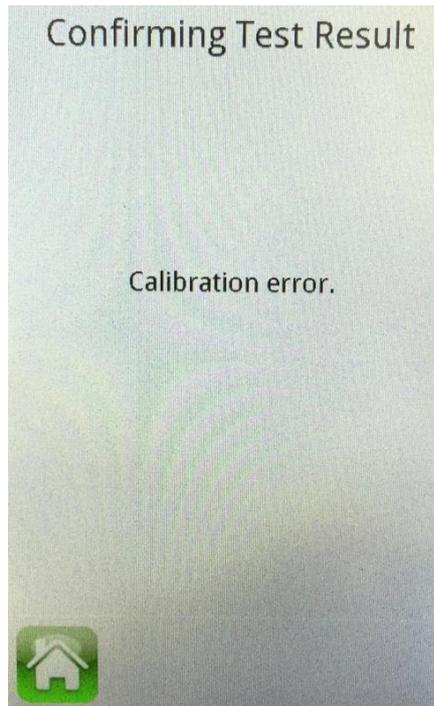


16. Print test record if necessary.

✧ Empty fields will not appear on the printout.

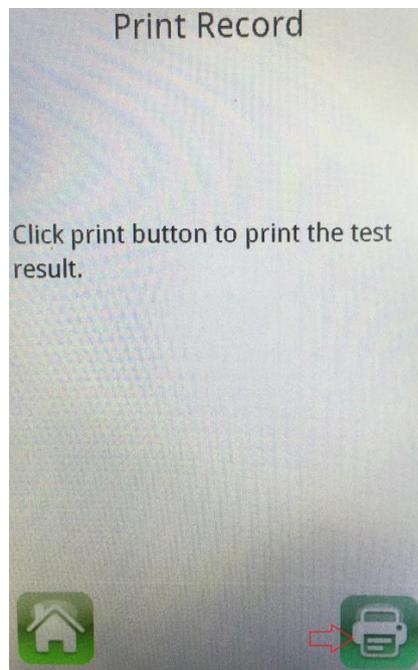

Function

1. Press “Function” icon to enter the function setting.

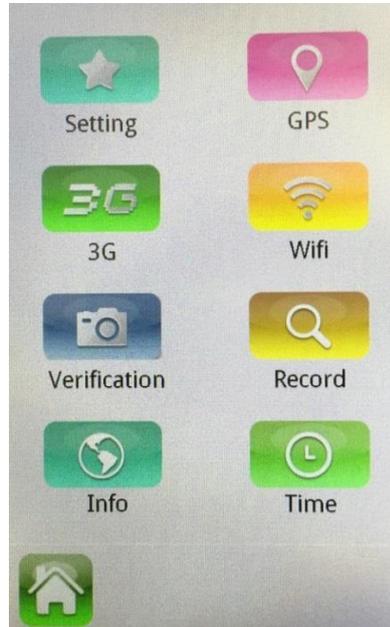

Setting

Setting backlight level, timer for stabilizing result, auto backlight off time and auto off time.



GPS

1. Longitude and Latitude of location of the device will be shown.
 2. Longitude and Latitude of the test is stored and printed on the printout.
- NOTE:** GPS system only functioned 5-10 minutes after turn on the device. It can only receive signal in open area. If there is no GPS signal, the Longitude and Latitude will not be shown on the printout.



3G

3G Connection Setting





Wifi

Wifi Connection Setting



Verification

If the camera does not work or cannot focus correctly on the test cassette, or error message appears, insert the test cassette in verification setting to adjust the camera. Just press “Verification” button, and then insert a test cassette.

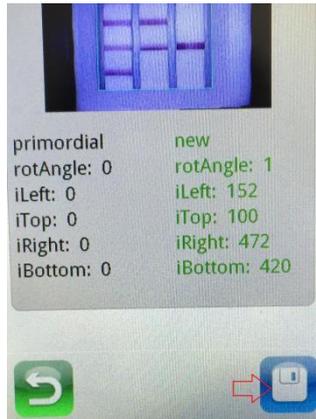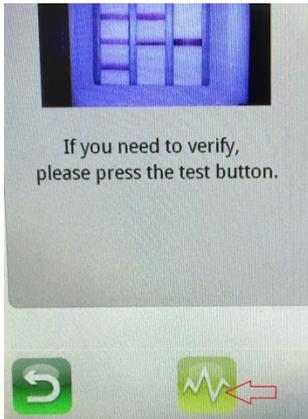

Record

1. Over 100,000 test records can be stored with the photo of the testees.
2. Select the record and view the details of the test record. Individual test record can be print.

3. Press  to search test records by fields and press  to confirm.



Info

Check the software version of the device.



Time

Date and time setting

Advanced Setting



Only legitimated persons with password are allowed for advanced setting.
Please refer to Advanced Setting Instruction.

PRINTER

An attachable built-in thermal printer is used for the device.

INSERT PAPER

Pull the handle to open the printer cassette and inset thermal paper. When inserting a new roll of paper, ensure glossy side of the paper is kept close to dentation.

NOTE: If nothing can be printed, change the side of the paper and try again.

STANDARD TERMAL PAPER SIZE

6cm X 500cm